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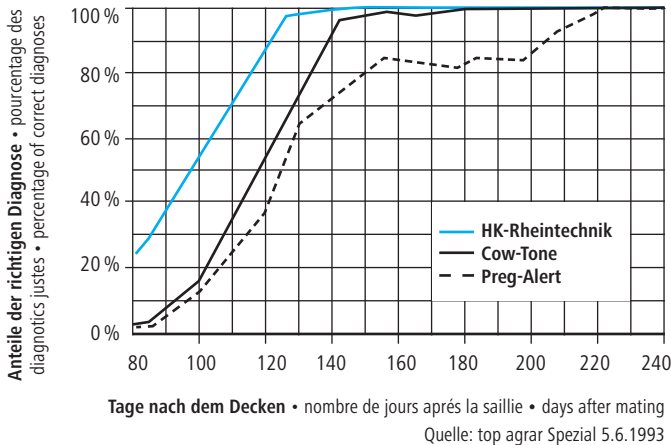
HK Pregnancy Tester
for Cows
Operating Instructions

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
The probe should be placed in the stifle fold in front of and above the udder on the abdominal wall and directed to the animals opposite side, between the hip-bone and the rear-rip

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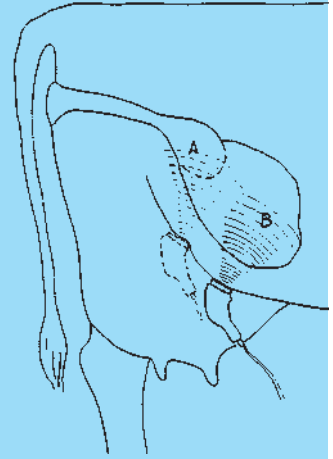Results of a pregnancy diagnosis test from cows between 80 and 240 days after pregnancy day.

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Diagnosis spot and direction of the probe in an early (A) and later (B) stage of pregnancy.

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EN HK pregnancy diagnosis unit for cows

With growth of the fetus and increase of uterine fluid, the uterus considerably enlarges as pregnancy progresses. In the 3rd to 4th month it will lower into the abdominal cavity. This is the best time for using the HK pregnancy detector. Diagnosis is possible from the 90th day, although this is affected by animal size, but is optimal from the 120th day. A clear audible continuous tone indicates a pregnant animal.

1. Operating-test

After activating the power switch on upper panel of the detector, apply some contact-oil on the face of the probe. Next, place the metal test probe against the probe face. If this produces a clear continuous tone and if the pregnancy control lamp lights up red continuously, the pregnancy tester is in working order and ready for use. Pay attention to the battery power control. If the red light is on, the battery must be changed very soon. The battery is completely empty if no light is on and must be changed immediately.

2. Contact-oil

The contact oil ensures airless contact between the animal's skin and the face of the probe. Before each diagnosis, apply a liberal amount of oil on the face of the probe. With the exception of mineral oils, all clean oils are suitable for this purpose. Paraffin oil, for example, has good properties. Do not handle with any used oil.

3. Diagnosis-spot and scanning-direction of probe

Animals to be diagnosed should always be tested on the right side. The optimal diagnosis spot is situated in the stifle fold in front of and above the udder on the abdominal wall (see Figure 1). The probe has to be directed to the animal's opposite side at an angle of approx. 45°. By moving the probe slightly forward and backward in an angle of approx. 30°, the ultrasonic waves scan the abdominal region, including the uterus where uterine fluid will reflect the ultrasonic waves. The Figure 1 and 2 demonstrates the placement of the probe.

4. Diagnosis

A clear audible continuous tone that lasts for 1-2 seconds indicates a pregnant animal. No signal, even after placing the probe more than once and using a good quantity of oil, indicates a non-pregnant animal.

5. Diagnosis precision

Diagnosis precision is optimal from the 120th to the 240th day of pregnancy. However, the following aspects should be taken into consideration:

- The animal to be tested should stand in a natural position.
- If the probe is moved too much backwards, the ultrasonic waves can be reflected by the fluid of a full bladder or the hip-bones (pelvis) giving a false positive result.
- Apply fresh oil on the face of the probe for each new placing.
- In case of negative result, repeat testing at a later date. This gives additional certainty. Gain some experience of testing by using cows which are known to be pregnant or not pregnant respectively.

SWITCH OFF POWER AFTER USE!

6. Changing battery

To change battery, open battery-compartment on the bottom of the tester. Therefore, push in the two lateral placed snap locks simultaneously with thumb and forefinger and take off the lid with the other hand. Use Alkaline batteries only (4 x 1,5 Volts Mignon). Other battery types diminish performance and should not be used.

7. Maintenance and care of detector

After each use, probe and instrument should be cleaned carefully. The probe can be cleaned with warm soaped water, the instrument with a dry clean cloth only. The tester should be kept in a clean dry place. Do not drop or mistreat device, as this risks to damage the electronics and the probe.

Careful maintenance and cleanliness will enhance the durability and performance of this instrument!

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Warranty

Albert Kerbl GmbH grants you an 2-year guarantee on this article in accordance with our general guarantee conditions. These general warranty conditions can be found at www.kerbl.com/guarantee. We will also be happy to send you the warranty conditions by post. Your statutory warranty rights remain unaffected by our guarantee.

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CE-conformity declaration

Albert KERBL GmbH hereby declares that the product / device described in these instructions complies with the fundamental requirements and other relevant stipulations and regulations. The CE mark confirms compliance with the Directives of the European Union. The declaration of conformity can be viewed by visiting www.kerbl.com

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Electronic scrap

Disposing of this device after its service life is the responsibility of the operator. Please consult the valid national regulations. The device must not be disposed of in household waste. In accordance with the stipulations of the EU Directive on the Disposal of Electrical and Electronic Devices, the device can be disposed of free of charge at the local waste collection or recycling centre. Alternatively, it can be returned to retailers who offer a collection service. The proper disposal helps to ensure environmental protection and prevents any adverse effects on human health and the environment.

Albert Kerbl GmbH

Felizenzell 9 84428 Buchbach, Germany www.rheintechnik.de
info@kerbl.com +49 8086 933-100

Kerbl UK Ltd

Lands End Way Oakham, Rutland LE15 6RF, UK
www.kerbl.co.uk